



# DOCPORT installation

1. Open the x86 or the x64 folder depending on your Windows version (32 or 64bits)

If you need to find your Windows version:

## Windows 10 and Windows 8.1

1. Select the Start button, then select Settings > System > About.

[Open About settings](#)

2. At the right, under Device specifications, see System type.

## Windows 7

1. Select the Start button, right-click Computer, and then select Properties.
2. Under System, see the system type.

2. Run the dplnst.exe file to install the camera drivers

3. Install the allSIGN.exe file located on the main folder

The allSign icon will be available on your Windows bar to modify the capture button

**Note that DocPort is no longer manufactured and there are no updates to software and drivers**

## Win 11

Install like Windows 10. System may say that your driver has an issue. Check Device Manager. Disable Core Isolation.

Select Start , enter 'Core Isolation' in the taskbar, and select Core Isolation from the list of results to open the Windows security app. **On the Core isolation page, turn off the toggle for Memory integrity.** You might need to restart your device.

The DocPort camera should be working properly now.